

PE anti-human CD58


Analyte Specific Reagent. Analytical and performance characteristics are not established.

Catalog# / Size	981702 / 500 μL
Clone	TS2/9
Other Names	Lymphocyte function-associated antigen 3, LFA-3
lsotype	Mouse IgG1, κ
Description	CD58, also known as lymphocyte function-associated antigen 3 (LFA-3), is a 45-70 kD cell surface protein that is a member of the immunoglobulin superfamily. Alternative splicing of CD58 gives rise to transmembrane and glycosylphosphatidylinositol (GPI)-anchored forms on cell surface. CD58 is expressed on both hematopoietic and non-hematopoietic cells including B cells, T cells, monocytes, erythrocytes, endothelial cells, epithelial cells, and fibroblasts. High levels are observed on memory T cells and dendritic cells. CD58 expressed on antigen presenting cells and target cells enhances T cell recognition via the binding of it's cognate ligand, CD2, on the T cell surface.

Product Details

Reactivity	Human				
Formulation	Phosphate-buffered solution, pH7.2, 0.09% sodium azide, 0.2% (w/v) BSA (origin USA), and a stabilizer.				
Preparation	The antibody was purified by affinity chromatography and conjugated with PE under optimal conditions.				
Concentration	400 μg/mL				
Storage & Handling	The antibody solution should be stored undiluted between 2°C and 8°C, and protected from prolonged exposure to light. Do not freeze .				
Application	Suggested for Flow Cytometry				
Disclaimer	WARNINGS AND PRECAUTIONS				
	 Use appropriate personal protective equipment and safety practices per universal precautions when working with this reagent. Refer to the reagent safety data sheet. This antibody contains sodium azide. Follow federal, state and local regulations to dispose of this reagent. Sodium azide build-up in metal wastepipes may lead to explosive conditions; if disposing of reagent down wastepipes, flush with water after disposal. All specimens, samples and any material coming in contact with them should be considered potentially infectious and should be disposed of with proper precautions and in accordance with federal, state and local regulations. Do not use this reagent beyond the expiration date stated on the label. Do not use this reagent if it appears cloudy or if there is any change in the appearance of the 				

reagent as these may be an indication of possible deterioration.6. Avoid prolonged exposure of the reagent or stained cells to light.

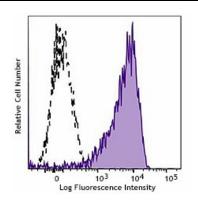

Typical results from human peripheral blood lymphocytes stained either with TS2/9 PE used at 5 μ L/test (filled histogram) or with an isotype control (open histogram).

Symbols Glossary*

Symbol	Meaning	Symbol Title	Symbol No.	Symbol	Meaning	Symbol Title	Symbol No.
REF	Catalog number	Catalogue number	5.1.6		Indicates the need for the user to consult the instructions for use.	Consult instructions for use	5.4.3
X	Indicates the temperature limits to which the medical device can be safely exposed.	Temperature limit	5.3.7	*	Indicates a medical device that needs protection from light sources.	Keep away from sunlight	5.3.2
X	Indicates the upper limit of temperature to which the medical device can be safely exposed.	temperature	5.3.6	Σ	Indicates the date after which the medical device is not to be used.	Use-by date	5.1.4
***	Indicates the medical device manufacturer.	Manufacturer	5.1.1	EC REP	Indicates the authorized representative in the European Community.	Authorized representative in the European Community	5.1.2
LOT	Indicates the manufacturer's batch code so that the batch or lot can be identified.	Batch code	5.1.5	IVD	Indicates a medical device that is intended to be used as an in vitro diagnostic medical device.	<i>In vitro</i> diagnostic medical device	5.5.1

* Symbol information is from EN ISO 15223-1:2016 Medical devices – Symbols to be used with medical device labels, labelling and information to be supplied – Part 1: General requirements

Not for resale. BioLegend will not be held responsible for patent infringement or other violations that may occur with the use of our products.

*These products may be covered by one or more Limited Use Label Licenses (see the BioLegend Catalog or our website, www.biolegend.com/ordering#license). BioLegend products may not be transferred to third parties, resold, modified for resale, or used to manufacture commercial products, reverse engineer functionally similar materials, or to provide a service to third parties without written approval of BioLegend. By use of these products you accept the terms and conditions of all applicable Limited Use Label Licenses.

8999 BioLegend Way, San Diego, CA 92121 www.biolegend.com Toll-Free Phone: 1-877-Bio-Legend (246-5343) Phone: (858) 768-5800 Fax: (877) 455-9587