

APC/Fire™ 750 anti-human CD36

Analyte Specific Reagent. Analytical and performance characteristics are not established.

Catalog# / Size 986304 / 500 μL

Clone 5-271

Other Names GPIIIb, GPIV

Isotype Mouse IgG2a, κ

Description CD36 is an 85 kD integral membrane glycoprotein, also known as GPIIIb, or GPIV. It is

expressed on various epithelial and endothelial cells as well as erythrocytes, platelets, macrophages/monocytes, and some macrophage-derived dendritic cells. CD36 functions as a scavenger receptor, binding thrombospondin, long chain fatty acids, oxidized LDL, collagen

type I, IV, and V as well as apoptotic cells.

Product Details

Reactivity Human

Formulation Phosphate-buffered solution, pH7.2, containing True-Stain Monocyte Blocker™, 0.09% sodium

azide, 0.2% (w/v) BSA (origin USA), and a stabilizer.

Preparation The antibody was purified by affinity chromatography and conjugated with APC/Fire™ 750 under

optimal conditions.

Concentration 100 μg/mL

Storage & Handling The antibody solution should be stored undiluted between 2°C and 8°C, and protected from

prolonged exposure to light. Do not freeze.

Application Suggested for Flow Cytometry

Disclaimer WARNINGS AND PRECAUTIONS

1. Use appropriate personal protective equipment and safety practices per universal precautions when working with this reagent. Refer to the reagent safety data sheet.


This antibody contains sodium azide. Follow federal, state and local regulations to dispose of this reagent. Sodium azide build-up in metal wastepipes may lead to explosive conditions; if disposing of reagent down wastepipes, flush with water after disposal.

3. All specimens, samples and any material coming in contact with them should be considered potentially infectious and should be disposed of with proper precautions and in accordance with federal, state and local regulations.

4. Do not use this reagent beyond the expiration date stated on the label.

5. Do not use this reagent if it appears cloudy or if there is any change in the appearance of the reagent as these may be an indication of possible deterioration.

6. Avoid prolonged exposure of the reagent or stained cells to light.

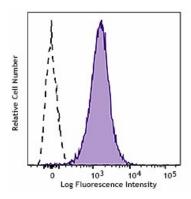

Typical results from human peripheral blood platelets stained either with 5-271 APC/Fire™ 750 used at 5 µL/test (filled histogram) or with an isotype control (open histogram).

Symbols Glossary*

Symbol	Meaning	Symbol Title	Symbol No.	Symbol	Meaning	Symbol Title	Symbol No.
REF	Catalog number	Catalogue number	5.1.6	(i	Indicates the need for the user to consult the instructions for use.	Consult instructions for use	5.4.3
1	Indicates the temperature limits to which the medical device can be safely exposed.	Temperature limit	5.3.7	漆	Indicates a medical device that needs protection from light sources.	Keep away from sunlight	5.3.2
K	Indicates the upper limit of temperature to which the medical device can be safely exposed.	temperature	5.3.6	Ω	Indicates the date after which the medical device is not to be used.	Use-by date	5.1.4
	Indicates the medical device manufacturer.	Manufacturer	5.1.1	EC REP	Indicates the authorized representative in the European Community.	Authorized representative in the European Community	5.1.2
	Indicates the manufacturer's	Batch code	5.1.5		Indicates a medical device that is	In vitro diagnostic	5.5.1
LOT	batch code so that the batch or lot can be identified.			IVD	intended to be used as an in vitro diagnostic medical device.	medical device	

^{*} Symbol information is from EN ISO 15223-1:2016 Medical devices – Symbols to be used with medical device labels, labelling and information to be supplied – Part 1: General requirements

Not for resale. BioLegend will not be held responsible for patent infringement or other violations that may occur with the use of our products.

*These products may be covered by one or more Limited Use Label Licenses (see the BioLegend Catalog or our website, www.biolegend.com/ordering#license). BioLegend products may not be transferred to third parties, resold, modified for resale, or used to manufacture commercial products, reverse engineer functionally similar materials, or to provide a service to third parties without written approval of BioLegend. By use of these products you accept the terms and conditions of all applicable Limited Use Label Licenses.

8999 BioLegend Way, San Diego, CA 92121 www.biolegend.com Toll-Free Phone: 1-877-Bio-Legend (246-5343) Phone: (858) 768-5800 Fax: (877) 455-9587